

23G ELEMENT CHANGE TO FULL-FIT

Beginning 4/18/12, the 23G Reserve Osmosis (RO) machine will be manufactured with full-fit membrane elements. This is a product improvement on the 23G. The use of full-fit elements is designed to minimize/reduce stagnate areas between the outside of the traditional membrane element and the housing.

The new element is still considered a spiral-wound membrane element. However, it does not utilize any type of tape outer wrap or hard, fiberglass shell, nor does it have the traditional brine seal. The membrane material is contained with a mesh outer cover that will allow the material to expand to meet the inside of the housing, eliminating the potential for a stagnant area (and resulting biological growth) that could exist when using tape or fiberglass wrapped elements.

The part number for the new full-fit element is 3027262. For existing 23G RO machines, an upgrade kit will be necessary to complete the first change-over. The part number for the kit is 3027594. It includes the element, special membrane - to - end cap adapters (2) and instructions. The existing membrane, OS1229398, will no longer be available as a replacement element.

The 23G RO serial number with the first set of new full-fit elements is 1298529, dated 4/18/12.

23G With TONKAFLO® Centrifugal Pumps

Mar Cor Purification discontinued manufacturing the 23G with the TONKAFLO pumps early in 2008. Facilities that have not updated their 23G with a Grundfos pump/motor should not replace their elements with the full fit membrane elements described above due to the possibility of fouling which decreases the permeate water production. The replacement membrane element for a 23G with the Tonkaflo® pump/motor is P/N OS1119067.

Contact Technical Support for any additional questions or product expertise on the 23G machine.

TONKAFLO® is a registered trademark of GE Osmonics, Inc.